

## WHAT DO THE LIGHTS MEAN ON THE PERFECTDRY LUX?



**Flashing blue light:**  
the dryer is active

---



**Light is off:**  
drying complete

---



**Solid blue light:** stand-by mode.  
This occurs briefly after the cover is closed and then the light goes out

---



# WIDEX PERFECTDRY LUX

ELECTRONIC DRYING STATION

**WIDEX**

SOUND LIKE NO OTHER

**WIDEX PerfectDry Lux electronic drying station is a perfect way to care for your hearing aids and earmoulds.**

**CONVENIENT**

PerfectDry Lux drying station is small and portable – allowing you to easily care for your hearing aids. Perfect Dry Lux is compatible with all Widex hearing aids, including WIDEX MOMENT™.

**SAFE AND HYGIENIC**

PerfectDry Lux drying station uses ultraviolet (UV) light, making it completely hygienic and reducing the risk of ear infections. UV lights shut off automatically when the device is opened, resulting in no risk to your vision.





## PerfectDry Lux

**USER FRIENDLY**

**CLEANING AND DRYING**

With 4 UV light sources, PerfectDry Lux drying station has a 360° approach — cleaning and drying hearing aids in 45 minutes. Simply place your hearing aids in the device with the battery doors opened, close the lid and press the start button — a blue light will appear. The device will automatically shut off when the 45-minute drying cycle is complete.