


THE CHARGER AT A GLANCE

Thank you for choosing this product. Your new WIDEX **MOMENT™** mRIC CHARGE n CLEAN charger is intended to detect, charge and maintain your WIDEX **MOMENT™** mRIC MRR2D hearing aid.


The charger consists of a charging unit for each hearing aid, a maintenance compartment for the receivers and three indicator lights that show the charging status of the hearing aids and the status of the maintenance cycle.

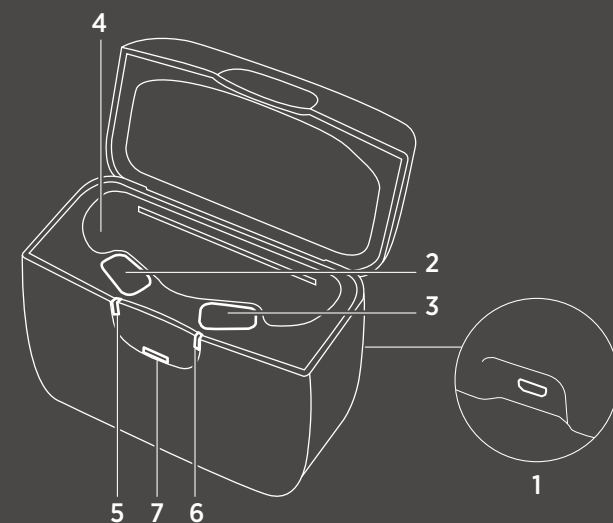

1. Micro USB Port
2. Left hearing aid charging well
3. Right hearing aid charging well
4. Receivers maintenance compartment
5. Left hearing aid charge indicator light
6. Right hearing aid charge indicator light
7. Maintenance indicator light

HOW TO GET STARTED

1. Connect the cable to the charger via the Micro USB port.
2. When the charger is connected to the power supply, the light indicators flash white 5 times and then turn off.

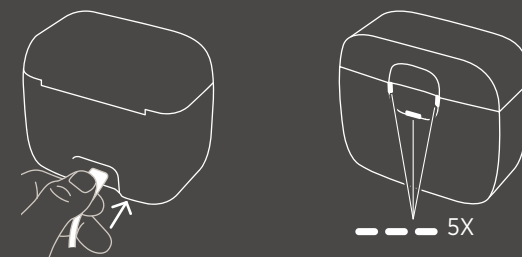


3. Gently place your hearing aids in their respective charging wells and the receivers in the maintenance compartment; then **close the lid**.




4. The charge and maintenance cycles run automatically (Full charge cycle 4 h, full maintenance cycle 30 min.).

CHARGING INDICATORS


While the charger is detecting the hearing aid, the light indicator turns solid yellow:

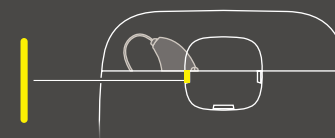


Once the charger has detected the hearing aid, the charging will begin. The light indicator will flash green to indicate that the hearing aid is low on battery and **less than 40% charged**:

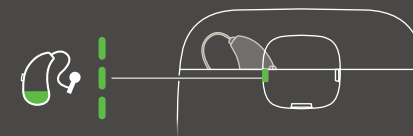

The light indicator will glow green to indicate that the hearing aid is **more than 40% charged**:

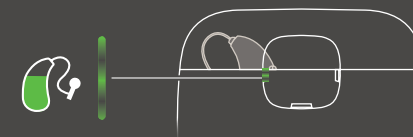


The light indicator will turn solid green to indicate that the hearing aid is **100% charged**:

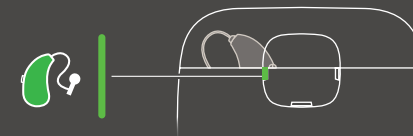


MAINTENANCE INDICATORS


The light indicator will flash white to indicate that the maintenance cycle is in progress (UV-C treatment/drying):

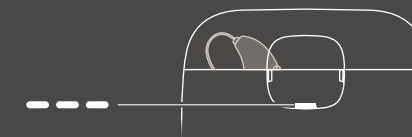

The indicator light turns solid white to indicate the end of the maintenance cycle:

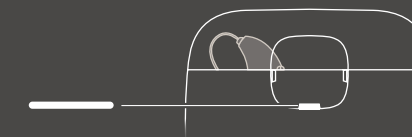


If the lid is opened during the maintenance cycle, the cycle is paused but the charging continues. The maintenance cycle will restart when the lid is closed.

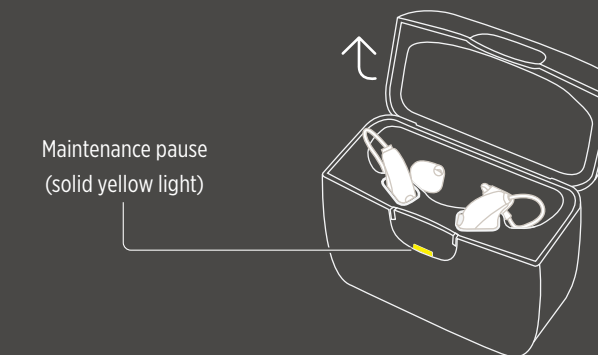

OVERVIEW OF LIGHT INDICATORS


CHARGE

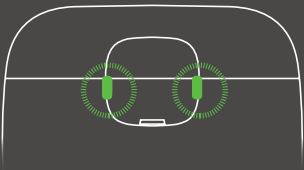

Solid yellow 	The charger is detecting the hearing aid.
Flashing green 	The hearing aid is charging, low battery, less than 40%.
Glowing green 	The hearing aid is charging, battery more than 40%.
Solid green 	The hearing aid is fully charged, battery 100% charged.

One indicator flashing red 	An error occurred during charging, and the charging has stopped.
Both indicators flashing red 	An error occurred during charging, and the charging has stopped.
Flashing yellow 	The temperature of the hearing aid has become too high during charging, and charging is paused.

MAINTENANCE

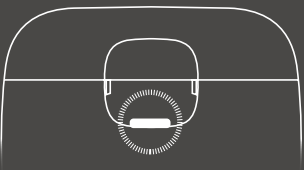

Solid yellow 	The maintenance cycle is paused, but the charging cycle continues.
Flashing white 	The maintenance cycle is in progress (drying/UV-C).
Solid white 	The maintenance cycle is complete.

Flashing red 	An error occurred during the maintenance, the cycle has stopped.
Flashing yellow 	The maintenance cycle is paused due to overheating (fan still running).



WWW.WIDEX.PRO

QUICK GUIDE



WIDEX **MOMENT™** mRIC CHARGE n CLEAN

For more information, consult the User Instructions that came with your **MOMENT™** mRIC CHARGE n CLEAN charger, or see the tutorial on how to use the charger in the WIDEX **MOMENT™** app.



Printed 2020-01. 9 502 5549 001 #01 - WIDEX MOMENT™ CHARGE n CLEAN Quick Guide

Subject to change with no further notice.

Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc. Android is a trademark of Google Inc. Google Play and the Google Play logo are trademarks of Google Inc.

WIDEX

SOUND LIKE NO OTHER